



Use the Eswab
Collection and
Transport System

Remove the swab,
taking care not to
touch the tip of the
swab or lay it down.

Hold the swab in
your hand, placing
your thumb and
forefinger in the
middle of the swab
shaft.

Gently swab the
posterior pharynx,
tonsils and inflamed
areas. Avoid the
tongue, cheeks, and
teeth when
collecting the
specimen.

Insert the swab into
the transport
medium

Break off at the pre-
scored line

Replace the cap onto
the tube and close
tightly.

Specimen transport and storage temperature (°C): 15-30 °C up to 48 hours / 2-8 °C up to 6 days